# Gerzer®/Gerzer® 2

Geradehalter | Posture Correctur  
Correttore di postura | Correcteur  
de posture | Corrector postural

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Temperaturbegrenzung

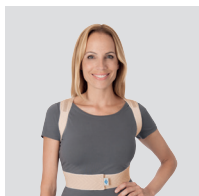
## ◆ Content


|                                   |             |
|-----------------------------------|-------------|
| Description                       | <b>EN-2</b> |
| Indication                        | <b>EN-2</b> |
| Function                          | <b>EN-2</b> |
| Reimbursement Code                | <b>EN-2</b> |
| Medical Device                    | <b>EN-2</b> |
| Safety                            | <b>EN-3</b> |
| REF/Sizes                         | <b>EN-4</b> |
| Application/Adjustment            | <b>EN-5</b> |
| Comfort/Hygiene                   | <b>EN-5</b> |
| Material Gerzer                   | <b>EN-5</b> |
| Material Gerzer 2                 | <b>EN-5</b> |
| Produkt Care/Cleaning/Maintenance | <b>EN-6</b> |
| Storage/Disposal                  | <b>EN-6</b> |
| CE-Conformity                     | <b>EN-6</b> |
| Legend to Symbols                 | <b>EN-6</b> |

## ◆ Description

When correctly positioned the Gerzer/ Gerzer 2 pulls the shoulders of the patient backwards and supports him or her in the correct posture. The broad, functional knitted fabric distributes the pressure over the surface and prevents friction and digging in.

Gerzer

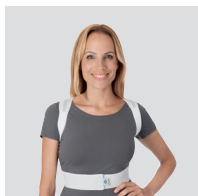Gerzer 2

## ◆ Indication

- Posture weakness of the shoulders

## ◆ Function

- Preventative elastic bandage to support posture

## ◆ Reimbursement Code

DE: HMV-Nr.: 23.13.01.0006

AT: Tarif-Nr.: 31012 011 (Gerzer)

AT: Tarif-Nr.: 31012 026 (Gerzer 2)

## ◆ Medical Device

**MD**

The Gerzer® / Gerzer® 2 is a medical device.

## ◆ ⚠ Safety

- Before this device is used, the orthosis has to be individually adapted to the patient by an experienced orthopedic technician and the patient has to be trained in the orthosis' use according to the doctor's instructions.
- Incorrect selection, adaptation, application, and/or failure to monitor the correct fit of this product may cause health damage.
- Check orthosis for damage (such as loose connections, fasteners, belts, etc.) each time it is put on.
- Do not remove the textile label from textile products.
- In case you suspect any performance malfunctions (e.g. an apparent reduction or lack of effect), immediately call on your doctor or orthopedic technician.
- All serious incidents that occur in connection with this product have to be reported to the Wilhelm Julius Teufel GmbH and the competent authority of the member state in which you are established. A „serious incident“ is any incident that caused, may have caused, or may cause, directly or indirectly, one of the following consequences:
  - a) the death of a patient, user, or other persons
  - b) temporary or permanent health deterioration of a patient, user or other persons
  - c) a serious public health threat.
- Never wear the orthosis on broken skin.
- The orthosis must be checked for proper function, correct fit and potential damage by a doctor or orthopedic technician at regular intervals. The timing of these intervals has to be defined individually for each patient.
- Should the product have been subject to disproportionate stress, it must be checked for potential damage by an orthopedic technician before further use.
- Harmful pressure caused by the orthosis may cause skin injuries or disrupt blood flow and must be avoided.
-  This product is intended for use on one person. It may not be reused on another person.

## ◆ REF/Sizes

| Gerzer® Posture Corrector |      |                            |      |               |       |
|---------------------------|------|----------------------------|------|---------------|-------|
| REF                       | Size | Underbust Circumference/cm |      | Clothing Size |       |
|                           |      | Min.                       | Max. | Women         | Men   |
| 40.386.001.00             | 1    | 54                         | 62   | 32            |       |
| 40.386.002.00             | 2    | 60                         | 69   | 34-36         | 34    |
| 40.386.003.00             | 3    | 66                         | 76   | 38-40         | 36-38 |
| 40.386.004.00             | 4    | 72                         | 83   | 42-44         | 40-42 |
| 40.386.005.00             | 5    | 80                         | 92   | 46-48         | 44-46 |
| 40.386.006.00             | 6    | 88                         | 101  | 50-52         | 48-50 |
| 40.386.007.00             | 7    | 96                         | 110  | 54            | 52-56 |

| Gerzer® 2 Posture Corrector |      |                            |      |               |       |
|-----------------------------|------|----------------------------|------|---------------|-------|
| REF                         | Size | Underbust Circumference/cm |      | Clothing Size |       |
|                             |      | Min.                       | Max. | Women         | Men   |
| 40.390.002.00               | 2    | 60                         | 69   | 34-36         | 34    |
| 40.390.003.00               | 3    | 66                         | 76   | 38-40         | 36-38 |
| 40.390.004.00               | 4    | 72                         | 83   | 42-44         | 40-42 |
| 40.390.005.00               | 5    | 80                         | 92   | 46-48         | 44-46 |
| 40.390.006.00               | 6    | 88                         | 101  | 50-52         | 48-50 |

## ◆ Application/Adjustment

1. Pull the shoulder loops over your arms so that the long backstitched seams of both loops are under your armpits (Fig. 1).

2. Pull the tensile bands tightly, crossing them over on your back (Fig. 2).

3. Draw the ends round to the front (Fig. 3).

4. Close the tensile bands in the front and proof the adequate tension for good fitting (Fig. 4).

## ◆ Comfort/Hygiene

Do not wear the orthosis directly on the skin, but instead over thin clothing (e.g. a t-shirt).

## ◆ Material Gerzer

- 75% Rayon
- 10% Elasthan
- 10% Cotton
- 5% Elastodiene

## ◆ Material Gerzer 2

- 76% Polyamide
- 24% Elasthane

## ◆ Produkt Care/ Cleaning/Maintenance

Please close the Velcro closures before washing.

Hand wash only.

Do not iron.

Do not tumble dry.

Do not bleach.

Do not dry-clean.

## ◆ Storage/Disposal

Product does not contain any environmentally hazardous substances. It can be disposed of with household refuse.

## ◆ CE-Conformity

This medical device complies with the requirements of the MDR, Regulation (EU) 2017/745.

## ◆ Legend to Symbols

Manufacturer

Date of Manufacture

Distributor

Importer

EC Authorized Representative

Lot Number

Part Number

Serial Number

UDI Data Matrix

Follow User Manual

Caution

CE-Marking According to European Union Council Regulation 2017/745 Concerning Medical Devices (MDR)

Single Patient, for Multiple Use

Store in a Dry Place, Protect from Moisture

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